SmartTENS
Wireless

Terapio⁺

Transcutaneous Electrical Nerve Stimulator

Wireless Pain Reliever

USER MANUAL

KTR-2493

DE

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ENGLISH

Read these instructions for use carefully and keep them for later use; be sure to make them accessible to other users and observe the information they contain.

TABLE OF CONTENT

Content of packaging	2
Getting to know your device	2
Important notes	3
Components of the device	7
Step-by-step operating instructions	8
Overview of the placement of the electrodes/pads	9
Switching the device on and off	10
Selection of the function	11
Setting the treatment intensity levels	11
Adjustment of treatment duration	11
Troubleshooting	12
Cleaning, storage and replacement parts	13
Warranty.....	14
The Paraphrase of Graphic Symbol	14
Executive Standards	15
Appendix EMC Declaration	16

Content of packaging

1. Wireless TENS Pain Reliever
2. 2x butterfly electropads
3. 1x remote control
4. 1x charging cable

Getting to know your device

What is a Wireless TENS Reliever unit and what can it do?

The wireless TENS Pain Reliever is a medical device that falls into the electrostimulation device category. It offers a wide range of functions for increasing general well-being, pain relief, maintaining physical fitness, relaxation, muscle revitalisation and combating tiredness.

TENS, or transcutaneous electrical nerve stimulation, relates to the electrical stimulation of the nerves via the skin. TENS is an effective non-pharmacological method of treating various types of pain that have a variety of causes. It has no side-effects if administered correctly. The method has been clinically tested and approved and can be used for simple self-treatment.

The pain-relieving or pain-suppressing effect is achieved by inhibiting the transference of pain to nerve fibres (caused mainly by high-frequency impulses) and by increasing the secretion of endorphins in the body. Their effect on the central nervous system reduces the sensation of pain.

The method is scientifically substantiated and approved as a form of medical treatment. Any symptoms that could be relieved using TENS must be checked by your GP. Your doctor will also give you instructions on how to carry out a TENS self-treatment regime.

TENS is clinically tested and approved to treat the following:

- Back pain, particularly in the lumbar/cervical spine area
- Sore joints (e.g. knee, hip and shoulder joints)
- Neuralgia
- Menstrual cramps in women
- Pain resulting from injury to the musculoskeletal system
- Pain caused by circulatory disorders
- Chronic pain with various causes.

Important notes

The device is not a substitute for medical consultation and treatment. Consult your doctor first if you are experiencing any pain or are suffering from an illness.

WARNING!

To avoid damage to health, we strongly advise against using this unit in the following situations:

- With implanted electrical devices (such as a pacemaker)
- In the case of metal implants
- If you use an insulin pump
- If you have a high temperature (e.g. > 39°C)
- If you have a known or acute cardiac arrhythmia, or disorders of the heart's impulse and conduction system
- If you suffer from a seizure disorder (e.g. epilepsy)
- If you are pregnant
- If you have cancer
- After an operation, if strong muscle contractions could affect the healing process
- The device must never be used near to the heart. The stimulation electrodes must not be placed on any part of the front ribcage (where the ribs and breastbone are located), especially not on the two large pectorals. This can increase the risk of ventricular fibrillation and induce cardiac arrest.
- On the skeletal skull structure, or around the mouth, throat or larynx
- In the neck/carotid artery area
- In the genital area

- On acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin—whether painful or not—reddened skin, rashes (e.g. allergies), burns, bruises, swellings, both open and healing wounds, and post-operative scars where the healing process could be affected)
- In humid environments (e.g. in the bathroom)/when bathing or showering
- Do not use after consuming alcohol
- If connected to a high-frequency surgical device
- In the case of acute or chronic diseases of the gastrointestinal tract
- The stimulation should not be applied above or through the head, directly onto the eyes, covering the mouth, to the front of the neck (especially not to the carotid artery), or with the electrode surfaces placed on the chest and upper back or across the heart.

Before using the device, consult your doctor if any of the following applies to you:

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thromboembolic conditions or recurrent malignant growths
- Any skin conditions
- Unexplained chronic pain in any part of the body
- Diabetes
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders)
- If you are receiving medical treatment
- In the event of complaints linked to stimulation treatment
- If you suffer from persistently irritated skin due to long-term stimulation at the same electrode site

Only use this unit

- On adults
- For the intended purpose and as specified in these instructions for use. Improper use can be dangerous.
- For external use
- With the original accessories supplied, which can be re-ordered. Failure to do so invalidates the warranty

PRECAUTIONS:

- Always pull firmly on electrodes to remove them from the skin to prevent injuries in the unusual case of highly sensitive skin.
- Keep the device away from sources of heat and do not use it in close proximity (approx. 1 m) to shortwave or microwave devices (e.g. mobile phones); doing so can result in unpleasant current peaks.
- Do not expose the device to direct sunlight or high temperatures.
- Protect the device from dust, dirt and humidity.
- Never immerse the device in water or other liquids.
- The device is suitable for self-treatment.
- For hygiene reasons, the electrodes may only be used on one person.
- If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.
- Switch off the unit or the respective channel first before removing or relocating electrodes in order to prevent unintentional stimulation.
- Do not modify electrodes (e.g. by cutting them).
- Do not use the device whilst asleep, driving a vehicle or operating machinery.
- Do not use whilst undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous.
- Ensure that no metallic objects (e.g. belt buckles or necklaces) come into contact with the electrodes during stimulation. If you are wearing jewellery or have piercings in the area to be treated (e.g. a navel piercing), these must be removed before using the device. Failure to do so could result in spot burns.
- Keep the device away from children.
- Do not entangle the electrode cables (including contacts) with headphones or other devices; do not connect the electrodes to other devices.
- Do not use the device whilst using other devices that transmit electrical impulses into your body.
- Do not use in the vicinity of highly flammable substances, gases or explosives.
- Do not use rechargeable batteries; always use the same battery types.

- During the initial few minutes, use the device while sitting or lying down to minimise the risk of injuries as a consequence of isolated cases of vagal responses (feeling of faintness). If you feel faint, immediately switch off the device, lie down and support the legs in an elevated position (approx. 5-10 min).
- Treatment of the skin with moisturising lotions or ointments beforehand is not recommended as this considerably increases the electrode wear and may cause unpleasant current peaks.
- This device is not intended for use by people with restricted physical, sensory (e.g. reduced sensitivity to pain) or mental skills or a lack of experience and/or lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device.
- If the adhesive capability of the adhesive electrodes decreases, replace them immediately. Only use the device again with new adhesive electrodes. Otherwise the unequal adhesion of the adhesive electrodes may lead to skin injuries. Replace the electrodes with new ones after having used them 20 times at most.

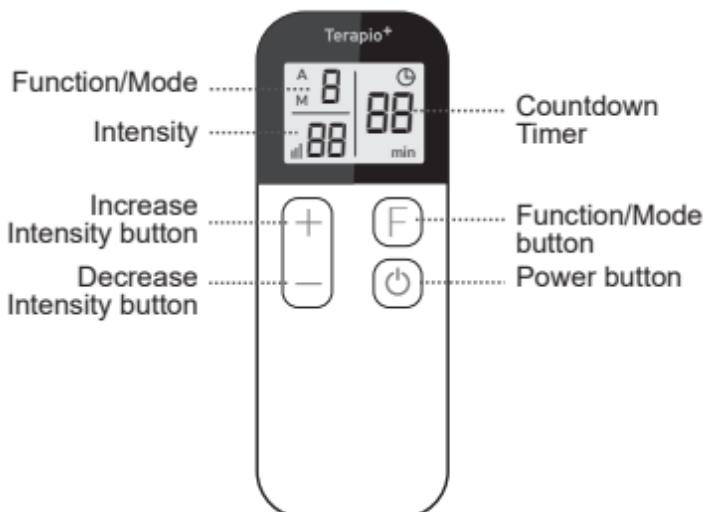
Damage

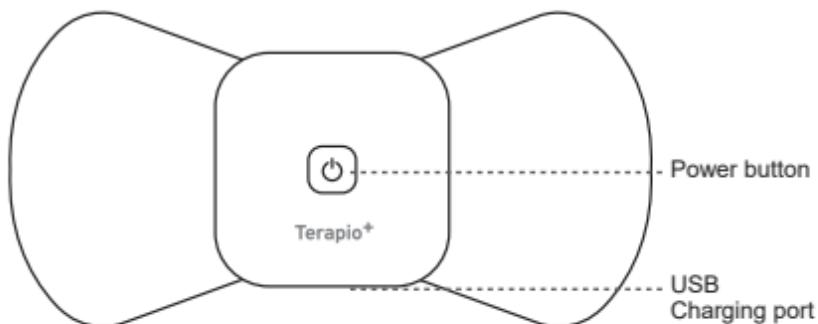
- If the device is damaged, do not use it; contact your retailer or the specified Customer Services address.
- In order to preserve the device's effectiveness, do not drop it or disassemble it.
- Check the device for signs of wear and tear or damage. If there are such signs or if the device has been used improperly, it must be returned to the manufacturer or retailer before further use.
- Switch the device off immediately if it is faulty or not working properly.
- Do not attempt to open and/or repair the device yourself. Repairs may only be carried out by Customer Services or authorised retailers. Failure to comply will result in voiding of the warranty.
- The manufacturer is not liable for damage resulting from improper or careless use.

Notes on handling batteries

- The device is powered by a lithium battery, which can be recharged via a USB cable
- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Choking hazard! Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- Risk of explosion! Never throw batteries into a fire.
- The first note states the battery can be recharged or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.

Components of the device


Operating instructions step-by-step

Charging of the unit

1. Locate the micro USB charging port located at the bottom of the unit.
2. Plug in the supplied charging cable into the unit and into any USB port.
3. Charge the unit for approximately two hours before using it.

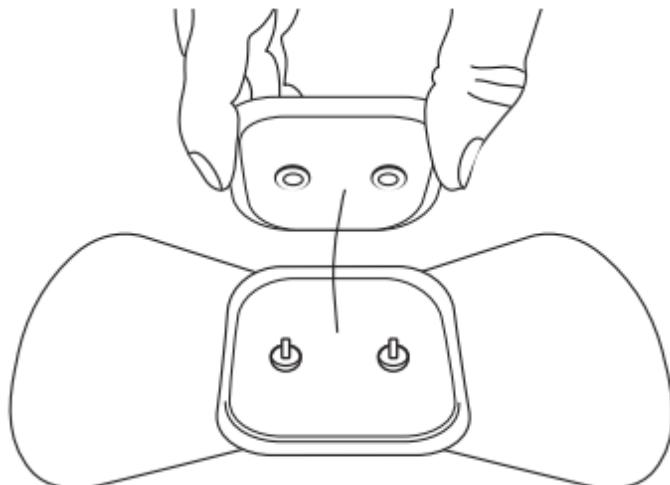Connecting the electropad to the main unit


The power unit attaches to the main unit via two metal press studs. Align the press studs, then push firmly until the control unit clicks into place.

Overview of the placement of the electrodes/pads

Proper preparation of the skin that is covered by the electrodes allows a better stimulation of the target tissue, prolongs the life of the electrodes and reduces the risk of skin irritation.


Prepare your skin at the spots where the electrode will be placed as follows:

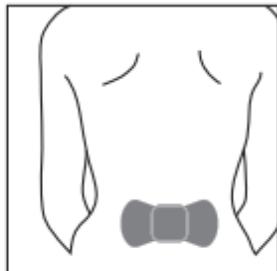1. Determine the spots where the electrode is to be placed.
2. Wash this area with mild soap and water (do not use alcohol). Rinse and dry carefully.
3. Trim excess body hair in this area with a pair of scissors (do not shave).
4. Treatment of the skin with moisturising lotions or ointments beforehand is not recommended as this considerably increases the electrode wear and may cause unpleasant current peaks, to supply the skin with a protective layer. Apply, let dry and attach the electrode as described in this manual. In this way you decrease the risk of skin irritation and prolong the life of your electrodes.
5. When you remove the electrodes, always pull them in the direction of the hair growth.
6. It may be helpful to apply skin lotion to the area of the electrode placement when you are not wearing the electrodes.

LOWER BACK

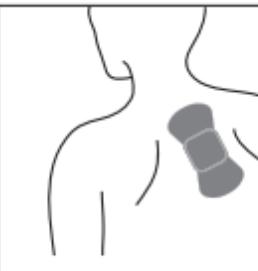Place the unit vertically on each side of your back in the lower back area.

UPPER BACK

Place the unit vertically on each side of your back in the upper back area.

SHOULDER

Place the unit on the top part of your shoulder.

STOMACH MUSCLES

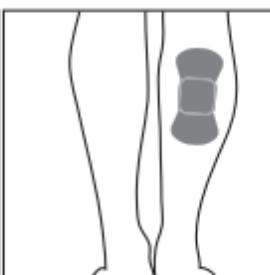
Place the unit vertically on your belly button.

FRONT OF THE THIGHS

Place the unit vertically on the front of the thigh muscle.

BACK OF THE THIGHS

Place the unit vertically on the back of the thigh muscle. Do not place pads too low on the leg, as this can lead to unpleasant contractions.

BUTTOCK

Place the unit vertically on the buttocks, halfway between the centre line and the side of your body.

CALF MUSCLES

Place the unit vertically on each calf muscle. Do not place the pads too low on the leg, as this lead to unpleasant contractions.

Switching the device on and off

1. Press and hold the power button on the unit to switch on the device.
2. A beep will sound and a green light will show.
3. Now press the power button on the remote control. Two beeps will sound to indicate that the remote and the device are paired.
4. The device turns off automatically after treatment time has elapsed.
5. Press the power button to switch off the device manually.

NOTE: Do not switch the device on until the electrodes have been connected correctly.

Selection of the function

The device offers 15 therapeutic programs divided into two functions:

- Automatic
- Manual

Upon activation the device is automatically set to automatic. The letter "A" displays in the upper left corner of the screen.

There are six modes in the automatic function. Press the "F" (function mode) button on the remote control quickly to cycle through combinations of pulses and find a mode that you prefer. Each time you switch Function or Mode the device intensity will reset to level "1".

To switch to manual, press and hold down the "F" button on the remote control. The letter "M" will display in the upper left corner of the screen. Press the "F" button briefly to cycle through the pulse modes. There are nine pulse modes in Manual function.

To return to the automatic function, press and hold down the "F" button once again.

Setting the treatment intensity levels

There are 16 steps of intensity. Press the "+" button on the remote control repeatedly to increase the intensity to a level you find comfortable.

If the intensity becomes too strong, press the "-" button to reduce the strength of the pulses. The device will beep with each step up or down in intensity.

Adjustment of treatment duration

Each session lasts for 15 minutes. The remaining minutes of the session are displayed in the upper right corner of the

screen on the remote control.

The remote control will automatically turn off after the 15 minutes have elapsed.

To begin another session, simply re-activate the remote control, or press the "+" intensity button on the device.

The maximum recommended number of consecutive sessions is four (60 minutes).

Troubleshooting

Before use you should check the device and accessories to avoid damage or malfunction. Some simple checks are:

1. Make sure that the unit is sufficiently charged and that the remote control's battery is full.
2. Ensure that the electropads are firmly inserted into the socket device.

The table below shows some problems that may occur occasionally. If you are unable to solve the problem yourself please contact your authorised dealer or customer service.

Problem	Cause	Solution
The device does not turn on	The batteries are not charged or running low	Recharge the unit
The device does not turn on, but generates electrical impulses	The batteries are flat, The electropad is defective or not properly connected.	Recharge the unit; reconnect the electropad or use a new one
	The duration of the treatment has elapsed	Switch off the device and then switch it back on again
The device does not turn on despite the unit being fully charged		Contact customer service

Cleaning, storage and replacement parts

Adhesive electrodes

- To ensure that the electrodes remain adhesive for as long as possible, clean them carefully with a damp, lint-free cloth or clean the underside of the electrodes under lukewarm, running water and pat dry with a lint-free cloth. Reapply the electrodes to the carrier foil following treatment.

Cleaning the device

- Remove the batteries from the device before cleaning.
- Clean the device after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution.
- Do not use any chemical or abrasive cleaning agents. Ensure that no water enters the device.

Reuse of the device

Once it has been properly prepared, the device can be used again. Preparation includes replacement of the treatment electrodes as well as cleaning the surface of the device using a cloth moistened with a mild soapy solution.

Storage

- Do not bend the electrodes.
- Disconnect the electropads when not in use
- Reapply the electrodes to the carrier foil after use.
- Store the device and accessories in a cool, well-ventilated space.
- Never place any heavy objects on the device.

Replacement parts

Order original electrodes and replacement parts from terapio.com

Warranty

This product is guaranteed for a period of one year from the date of purchase against mechanical and electrical manufacturing defects. There are no serviceable parts inside the device. Any attempted repair by unauthorised persons invalidates the warranty. In the unlikely event that you experience a problem, please return it to the retailer where you made the purchase, along with your receipt. This does not affect your statutory rights.

Contact customer support at terapio.com for any assistance.

The Paraphrase of Graphic Symbol

GRAPHIC SYMBOL	PARAPHRASE
LOT	Production Batch
REF	Product catalogue reference code
	Manufacturer (Directive 93/42/EEC amended by Directive 2007/47/EC)
	DATE OF MANUFACTURE. This symbol shall be accompanied by a date to indicate the date of manufacture.
	Caution
	Applied part of type BF

	European Authorized Representative
	Symbol for CE Mark. This symbol certifies that a product has met European Union consumer safety, health, or environmental requirements..
	CE marking with the Registration Number of the Notified Body. This denotes compliance with European Directive 93/42/EEC concerning medical devices.
	Level of protection against the insertion of solid bodies of size/diameter ≥ 12 mm and liquids in the presence of dripping water when tilted at 15° compared with product.s.
	Refer to instruction manual/booklet

Executive Standards

The product conforms to the following standards and laws:

1. IEC 60601-1:2005+A1:2012 Medical electrical equipment-Part 1: General requirements for basic safety and essential performance
2. IEC 60601-1-11:2015 Medical Electrical Equipment - Part 1-11: General Requirements For Basic Safety And Essential Performance - Collateral Standard: Requirements For Medical Electrical Equipment And Medical Electrical Systems Used In The Home Healthcare Environment
3. IEC 60601-2-10:2013 Medical electrical equipment - Part 2-10: Particular requirements for the safety of nerve and muscle stimulators
4. IEC 60601-1-2:2014 Medical Electrical Equipment - Part 1-2: General Requirements For Basic Safety And Essential Performance - Collateral Standard: Electromagnetic Disturbances - Requirements And Tests

Appendix EMC Declaration

The equipment is intended for use in the electromagnetic environment specified below.

The customer or the user of the EQUIPMENT should assure that it is used in such an environment.

The Transcutaneous Electrical Nerve Stimulators is suitable for use in a professional health care environment, not including areas where there are sensitive equipment or sources of intense electromagnetic disturbances, such as the RF shielded room of an imaging system magnetic resonance imaging, in operating rooms near active AF surgical equipment, electrophysiology laboratories, armored rooms or areas where short wave therapy equipment is used.

Danger

- Do not use the system around strong electric filed, lectromagnetic filed (e.g. MRI scan room) and mobile wireless communication devices. Using the device in an improper environment may cause malfunction or damage.
- The compliance with EMC and EMI regulation cannot be guaranteed by the use of modified cables or those which does not comply with the same standards under what the equipment was validated.
- The system must not be used adjacent or supported by other equipment. The recommendations of this manual must be followed.
- Do not use accessories, transducers, internal parts of components and other cables other than those previously specified by the manufacturer. This may result in increased emission or decreased electromagnetic immunity and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30cm to any

part of the ultrasound system, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

- To maintain basic safety in relation to electromagnetic disturbances during the expected service life, always use the system in the specified electromagnetic environment and follow the maintenance recommendation described in this manual.

The following tables provide information on compliance of the equipment according to the standard EN 60601-1- 2:2015.

Table 1 Compliance class

Emissions Test	Compliance	Electromagnetic Environment and Guidance
RF emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	The equipment is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Table 2- Compliance standards

Phenomenon	Basic Standard of	Immunity Test	Level of
Electrostatic discharge	IEC 61000-4-2	±8 KV contact ±2 KV, ±4 KV, ±8 KV,±15KV air	±8 KV contact ± 2 KV, ± 4 KV, ± 8 KV, ±15KV air
Radiated RF EM fields1	IEC 61000-4-3	3V/m 80 MHz-2.7 GHz 80% AM at 1 KHz	3V/m 80 MHz-2.7 GHz 80% AM at 1 KHz
Proximity fields from RF wireless communication equipment	IEC 61000-4-3	See table	See table
Electrical Fast/Transients bursts	IEC 61000-4-4	±1 KV 100 KHz repetition frequency	±1 KV 100 KHz repetition frequency
Conducted disturbances induced by RF fields.	IEC 61000-4-6	3V 0.15 MHz-80 MHz 6 Vm in ISM bands between 0 .15 MHz and 80 MHz 80% AM at 1KHz	3V 0.15 MHz-80 MHz 6 Vm in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1KHz
Rated power frequency magnetic fields	IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz

**Table 3- Test specifications for
ENCLOSURE PORT IMMUNITY to RF
wireless communications equipments**

Test Frequency (MHz)	Band (MHz)	Service	Modulation	Maximum Power (W)	Distance (m)	Immunity Test Level (V/m)
385	380-390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430-470	GMRS 460, FRS 460	FM ±5 KHz deviation 1KHz sine	2	0.3	28
710	704-787	LTE 13, 17 Band	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE 5 Band	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720	1700-1990	GSM 1800 , CDMA 1900, GSM 1900, DECT,LTE 1, 3, 4 ,25 Band, UMTS	Pulse modulation 217 Hz	2	0.3	28
1845						
1970						
2450	2400-2570	Bluetooth, WLAN 802.11 b/g/n, RFID 2450, LTE 7 Band	Pulse modulation 217 Hz	2	0.3	28
5240	5100-5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

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Electronics Co.,Ltd**
WellKang Ltd (www.CE-marking.eu)