**SmartTENS**

**Terapio<sup>+</sup>**

Transcutaneous Electrical Nerve and Muscle Stimulators

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# ENGLISH

Read these instructions carefully and keep them for later use.

## TABLE OF CONTENT

Content of packaging .....	2
Getting to know your device .....	2
Important notes .....	4
Components of the device .....	8
Step-by-step operating instructions .....	8
Overview of the placement of the electrodes/pads .....	9
Switching the device on and off .....	11
Selection of the function .....	11
Setting the treatment intensity levels .....	11
Adjustment of treatment duration .....	12
Troubleshooting .....	12
Cleaning, storage and replacement parts .....	13
Warranty.....	14
The Paraphrase of Graphic Symbol .....	15
Executive Standards .....	16
Appendix EMC Declaration .....	17

## Content of packaging

1. 3-in-1 TENS/EMS/MASSAGE Pain Reliever
2. 4 square electro pads
3. 4 square electro pads
4. 2x cables
5. 3 AAA batteries

## Getting to know your device

### What is a Smart TENS Pain Reliever unit (3-in-1 TENS/EMS/MASSAGE ) and what can it do?

The Smart TENS Pain Reliever unit (3-in-1 TENS/EMS/MASSAGE ) is a medical device that falls into the electrostimulation device category. It provides three basic functions suitable for combined operation:

1. Electrical stimulation for pain relief (TENS)
2. Electrical stimulation of muscle regeneration (EMS)
3. Electrical stimulation for relaxation and massage (FITNESS)

The unit also features two independent stimulation channels and four self-adhesive electrodes. It offers a wide range of functions for increasing general well-being, pain relief, maintaining physical fitness, relaxation, muscle revitalisation and combating tiredness.

TENS, or transcutaneous electrical nerve stimulation, relates to the electrical stimulation of the nerves via the skin. TENS is an effective non-pharmacological method of treating various types of pain that have a variety of causes. It has no side-effects if administered correctly. The method has been clinically tested and approved and can be used for simple self-treatment.

The pain-relieving or pain-suppressing effect is achieved by inhibiting the transference of pain to nerve fibres (caused mainly by high-frequency impulses) and by increasing the secretion of endorphins in the body. Their effect on the central nervous system reduces the sensation of pain.

The method is scientifically substantiated and approved as a form of medical treatment. Any symptoms that could be relieved using TENS must be checked by your GP. Your doctor will also give you instructions on how to carry out a TENS self-treatment regime.

### **Function #1: TENS is clinically tested and approved to treat the following:**

- Back pain, particularly in the lumbar/cervical spine area
- Sore joints (e.g. knee, hip and shoulder joints)
- Neuralgia
- Menstrual cramps in women
- Pain resulting from injury to the musculoskeletal system
- Pain caused by circulatory disorders
- Chronic pain with various causes.

### **Function #2: Electrical muscle stimulation (EMS) is a widespread and recognised method and has been used in sports medicine and rehabilitation for years.**

In sports and fitness, EMS is used to complement conventional muscle training, to increase the performance of muscle groups and to adjust physical proportions to achieve the desired aesthetic results.

There are two types of EMS application: one is for targeted strengthening of the muscles (activating application), and the other is to achieve a relaxing, restful effect (relaxing application).

The activating application involves:

- Muscle training to increase endurance and/or
- Muscle training to support the strengthening of specific muscles or muscle groups, and to achieve the desired changes to physical proportions.

The relaxing application involves:

- Muscle relaxation for easing muscle tension
- Improving symptoms of muscular fatigue
- Acceleration of muscle regeneration after high muscle performance (e.g. after a marathon).

## Function #3: Massage

Thanks to integrated massage technology, this medical device is also able to relieve muscle tension and combat fatigue with a program based on the sensation and effects of a real massage.

## Important notes

The device is not a substitute for medical consultation and treatment. Consult your doctor first if you are experiencing any pain or if you are suffering from an illness.

## WARNING!

To avoid damage to health, we strongly advise against using this unit in the following situations:

- With implanted electrical devices (such as a pacemaker)
- In the case of metal implants
- If you use an insulin pump
- If you have a high temperature (e.g. > 39°C)
- If you have a known or acute cardiac arrhythmia, or disorders of the heart's impulse and conduction system
- If you suffer from a seizure disorder (e.g. epilepsy)
- If you are pregnant
- If you have cancer
- After an operation, if strong muscle contractions could affect the healing process
- The device must never be used near to the heart. The stimulation electrodes must not be placed on any part of the front ribcage (where the ribs and breastbone are located), especially not on the two large pectorals. This can increase the risk of ventricular fibrillation and induce cardiac arrest.
- On the skeletal skull structure, or around the mouth, throat or larynx
- In the neck/carotid artery area
- In the genital area
- On acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin—whether painful or not—reddened skin, rashes (e.g. allergies), burns, bruises, swellings, both open and healing wounds and post-operative scars where the healing process could be affected)

- In humid environments (e.g. in the bathroom)/when bathing or showering
- After consuming alcohol
- If connected to a high-frequency surgical device
- In the case of acute or chronic diseases of the gastrointestinal tract.

In addition, the stimulation should not be applied above or through the head, directly onto the eyes, covering the mouth, to the front of the neck (especially not to the carotid artery), or with the electrode surfaces placed on the chest and upper back or across the heart.

### **Before using the device, consult your doctor if any of the following applies to you:**

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thromboembolic conditions or recurrent malignant growths
- Any skin conditions
- Unexplained chronic pain in any part of the body
- Diabetes
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders)
- If you are receiving medical treatment
- In the event of complaints linked to stimulation treatment
- If you suffer from persistently irritated skin due to long-term stimulation at the same electrode site

## **IMPORTANT**

### **Only use this unit**

- On adults
- For the intended purpose and as specified in these instructions for use. Improper use can be dangerous.
- For external use
- With the original accessories supplied, which can be re-ordered. Failure to do so invalidates the warranty.

## **PRECAUTIONS:**

- Always pull firmly on electrodes to remove them from

the skin to prevent injuries in the unusual case of highly sensitive skin.

- Keep the device away from sources of heat and do not use it in close proximity (approx. 1 m) to shortwave or microwave devices (e.g. mobile phones); doing so can result in unpleasant current peaks.
- Do not expose the device to direct sunlight or high temperatures.
- Protect the device from dust, dirt and humidity.
- Never immerse the device in water or other liquids.
- The device is suitable for self-treatment.
- For hygiene reasons, the electrodes may only be used on one person.
- If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.
- Switch off the unit or the respective channel first before removing or relocating electrodes to prevent unintentional stimulation.
- Do not modify electrodes (e.g. by cutting them).
- Do not use the device whilst asleep, driving a vehicle or operating machinery.
- Do not use whilst undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous.
- Ensure that no metallic objects (e.g. belt buckles or necklaces) come into contact with the electrodes during stimulation. If you are wearing jewellery or have piercings in the area to be treated (e.g. a navel piercing), these must be removed before using the device. Failure to do so could result in spot burns.
- Keep the device away from children.
- Do not entangle the electrode cables (including contacts) with headphones or other devices; do not connect the electrodes to other devices.
- Do not use the device whilst using other devices that transmit electrical impulses into your body.
- Do not use in the vicinity of highly flammable substances, gases or explosives.
- Do not use rechargeable batteries; always use the same battery types.
- During the initial few minutes, use the device while sitting or lying down in order to minimise the risk of injuries as a consequence of isolated cases of vagal responses (feeling of faintness). If you feel faint,

- immediately switch off the device, lie down and support the legs in an elevated position (approx. 5–10 min).
- Treatment of the skin with moisturising lotions or ointments beforehand is not recommended as this considerably increases the electrode wear and may cause unpleasant current peaks.
- This device is not intended for use by people with restricted physical, sensory (e.g. reduced sensitivity to pain) or mental skills or a lack of experience and/or lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device.
- If the adhesive capability of the adhesive electrodes decreases, replace them immediately. Only use the device again with new adhesive electrodes. Otherwise the unequal adhesion of the adhesive electrodes may lead to skin injuries. Replace the electrodes with new ones after having used them 20 times at most.

## **Damage**


- If the device is damaged, do not use it; contact your retailer or the specified Customer Services address.
- In order to preserve the device's effectiveness, do not drop it or disassemble it.
- Check the device for signs of wear and tear or damage. If there are such signs, or if the device has been used improperly, it must be returned to the manufacturer or retailer before further use.
- Switch the device off immediately if it is faulty or not working properly.
- Do not attempt to open and/or repair the device yourself. Repairs may only be carried out by Customer Services or authorised retailers. Failure to comply will result in voiding of the warranty.
- The manufacturer is not liable for damage resulting from improper or careless use.

## **Notes on handling batteries**

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Choking hazard! Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.

- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- Risk of explosion! Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.

## Components of the device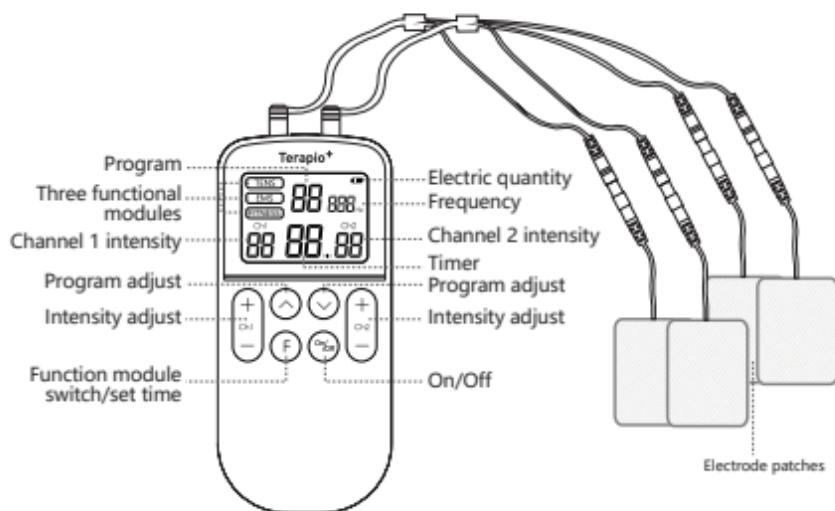


## Step-by-step operating instructions

### Insertion/changing of the batteries

1. Open the battery compartment at the back
2. Insert three AAA batteries (1.5 V) into the battery compartment; make sure that the symbols (+/- match).
3. Close the battery compartment.

### Connecting cables to the device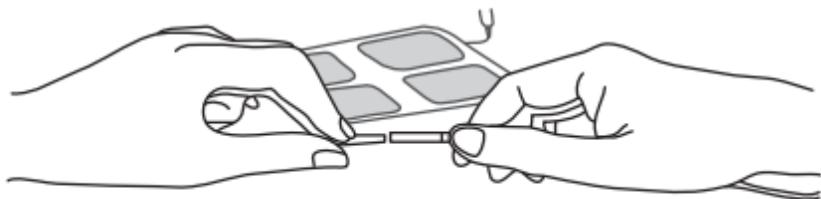


1. Connect the cable to the electrodes before placing them on the skin.
2. Plug the electrode cables into both channels.

## Overview of the placement of the electrodes/pads

Proper preparation of the skin that is covered by the electrodes allows a better stimulation of the target tissue, prolongs the life of the electrodes and reduces the risk of skin irritation.

After connecting the cable to the stimulator, prepare your skin at the spots where electrodes will be placed as follows:

1. Determine the spots where the electrodes are to be placed.
2. Wash this area with mild soap and water (do not use alcohol). Rinse and dry carefully.
3. Trim excess body hair in this area with a pair of scissors (do not shave).
4. Optionally, apply skin lotion, to the area to supply the skin with a protective layer. Treatment of the skin with moisturising lotions or ointments beforehand is not recommended as this considerably increases the electrode wear and may cause unpleasant current peaks. Apply, let dry and attach the electrode as described in this manual. In this way you decrease the risk of skin irritation and prolong the life of your electrodes.
5. When you remove the electrodes, always pull them in the direction of the hair growth.
6. It may be helpful to apply skin lotion to the area of the electrode placement when you are not wearing the electrodes.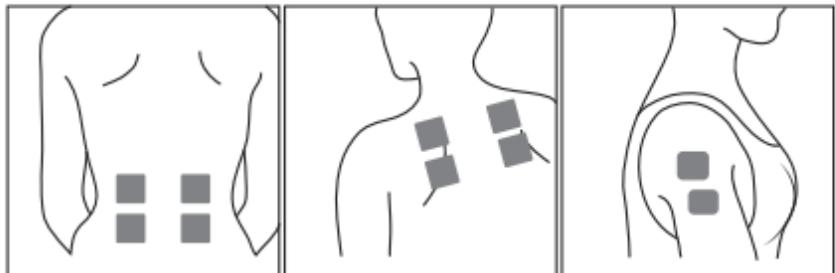

### **LOWER BACK**


Place a pair of pads vertically on each side of your back in the lower back area.

### **UPPER BACK**

Place a pair of pads vertically on each side of your back in the upper back area.

### **SHOULDER**

Place one half of the pad on the top part of your shoulder and the other half on the side.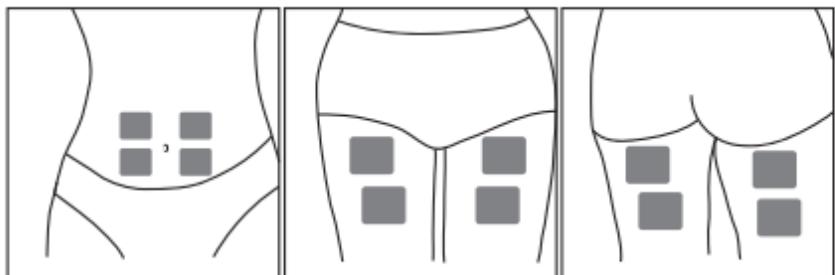

### **STOMACH MUSCLES**


Place each pair of pads vertically on each side of your belly button.

### **FRONT OF THE THIGHS**

Place each pair of pads vertically on each front of the thigh muscle.

### **BACK OF THE THIGHS**

Place each pair of pads vertically on each back of the thigh muscle. Do not place pads too low on the leg, as this can lead to unpleasant contractions.

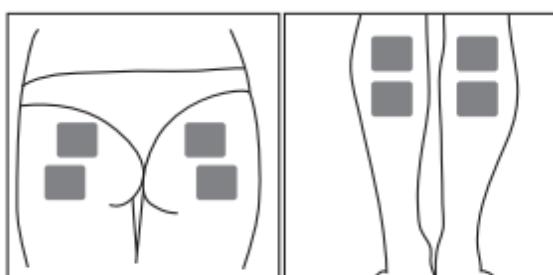### **BUTTOCK**

Place the pads vertically on the buttocks, halfway between the centre line and the side of your body.

### **CALF MUSCLES**

Place a pair of pads vertically on each calf muscle. Do not place the pads too low on the leg, as this lead to unpleasant contractions.

## **Switching the device on and off**

1. Press and hold the ON/OFF button to switch on the device.
2. The device turns off automatically after the treatment time has elapsed.
3. Press the ON/OFF button to switch off the device manually. The screen goes blank and the device turns off.

**NOTE:** Do not switch the device on until all electrodes and cables have been connected correctly.

## **Selection of the function**

The device offers 50 therapeutic programs divided into three functions:

- Electrical stimulation for pain relief (TENS)
- Electrical stimulation of muscle regeneration (EMS)
- Electrical stimulation for relaxation and massage (FITNESS)

1. Press the F key until the desired function has been selected (TENS/EMS/FITNESS)
2. Use the up and down arrows to find a suitable program. Programs differ in terms of pulse width and frequency. Select a program that you prefer.
3. We recommend alternating between programs throughout your treatment.

## **Setting the treatment intensity levels**

1. The intensity can be controlled individually for each channel.
2. To increase the intensity of each channel, repeatedly press the arrows on the CH1 and CH2 buttons

With two separately adjustable channels, the unit offers you the advantage of being able to set the intensity of the

impulses independently from each other for two treatment areas on the body, for example to cover both sides of your body or to evenly stimulate larger areas of tissue. You can also treat two separate areas of the body simultaneously instead of having to treat the individual areas in turn, which saves you time.

**NOTE:** If you stimulate the muscles of arms or legs, keep in mind that muscle contraction can lead to involuntary movement of limbs, through which you or other persons can be injured. Make sure that limbs are secured to prevent movement.

## Adjustment of treatment duration

1. By default a session lasts 15 minutes
2. To increase the duration of treatment, press and hold the F key.
3. You can choose between 15, 30 and 45 minutes
4. The unit turns off automatically after the treatment time has elapsed.

## Troubleshooting

Before use you should check the device and accessories to avoid damage or malfunction. Some simple checks are:

1. Make sure that the battery has sufficient power and that it is not corroded.
2. Ensure that the cables are firmly inserted into the socket device.

The table below shows some problems that may occur occasionally. If you are unable to solve the problem yourself please contact your authorised dealer or customer service.

PROBLEM	CAUSE	SOLUTION
The device does not turn on	The batteries are not charged or running low	Insert a new battery

The device turns on but then turns off	The batteries are not inserted correctly	Reinsert the batteries or replace them
The device does not turn on, but generates electrical impulses	The batteries are flat; the cord is defective or not properly connected.  The duration of the treatment has elapsed	Replace the batteries or cord; connect the cord properly  Switch off the device and then switch it back on again The device does not turn on despite new batteries
The device does not turn on despite new batteries		Contact customer service

## Cleaning, storage and replacement parts

### Adhesive electrodes

- To ensure that the electrodes remain adhesive for as long as possible, clean them carefully with a damp, lint-free cloth or clean the underside of the electrodes under lukewarm, running water and pat dry with a lint-free cloth. Before cleaning with water, remove the connection cables from the electrodes.
- Reapply the electrodes to the carrier foil following treatment.

### Cleaning the device

- Remove the batteries from the device before cleaning.
- Clean the device after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution.
- Do not use any chemical or abrasive cleaning agents.

Ensure that no water enters the device.

## Reuse of the device

Once it has been properly prepared, the device can be used again. Preparation includes replacement of the treatment electrodes as well as cleaning the surface of the device using a cloth moistened with a mild soapy solution.

## Storage

- Remove the batteries from the device if you will not be using it for a prolonged period of time. Leaking batteries may damage the device.
- Do not make sharp bends in the connection cables and electrodes.
- Disconnect the connection cables from the electrodes.
- Reapply the electrodes to the carrier foil after use.
- Store the device and accessories in a cool, well-ventilated space.
- Never place any heavy objects on the device.

## Replacement parts

Order original electrodes and replacement parts from [terapio.com](http://terapio.com)

## Warranty

This product is guaranteed for a period of one year from the date of purchase against mechanical and electrical manufacturing defects. There are no serviceable parts inside this device. Any attempted repair by unauthorised persons invalidates the warranty. In the unlikely event that you experience a problem, please return the device to the retailer where you made the purchase, along with your receipt. This does not affect your statutory rights.

Contact customer support at [terapio.com](http://terapio.com) for any assistance.

# The Paraphrase of Graphic Symbol

GRAPHIC SYMBOL	PARAPHRASE
<b>LOT</b>	Product batch number tag
<b>SN</b>	Product serial number tag
	Manufacturer
	Production date
	Precautions
	EU representative
 Danger	Warning against danger
	Applied part of type BF
	Class II equipment
	This product only for indoor use
	It means prohibition (the thing was not permitted to do)

	It means enforcement (the thing must be observed)
	Keep out of the sun
	Keep away from rain
	Contact its local authorities to determine the proper method of disposal of potentially bio hazardous parts and accessories.
	CE label and noticed body code
<b>IP22</b>	The dustproofing and waterproofing grade, that is, to prevent the solid object is greater than 12mm to invade, the inclination of 15 degrees can prevent dripping invasion still not cause harmful effects.
	This symbol is used to direct the user to refer to documentation for additional information regarding the system use or description.

## Executive Standards

The product conforms to the following standards and laws:

1. IEC 60601-1:2005+A1: 2012 Medical electrical equipment-Part 1: General requirements for basic safety and essential performance.
2. IEC 60601-1-11: 2015 Medical Electrical Equipment - Part 1-11: General Requirements For Basic Safety And Essential Performance - Collateral Standard: Requirements For Medical Electrical Equipment

And Medical Electrical Systems Used In The Home Healthcare Environment.

3. IEC 60601-2-10: 2013 Medical electrical equipment - Part 2-10: Particular requirements for the safety of nerve and muscle stimulators.
4. IEC 60601-1-2:2014 Medical Electrical Equipment - Part 1-2: General Requirements For Basic Safety And Essential Performance - Collateral Standard: Electromagnetic Disturbances - Requirements And Tests.

## Appendix EMC Declaration

The equipment is intended for use in the electromagnetic environment specified below.

The customer or the user of the EQUIPMENT should assure that it is used in such an environment.

The Transcutaneous Electrical Nerve and Muscle Stimulators is suitable for use in a professional health care environment, not including areas where there are sensitive equipment or sources of intense electromagnetic disturbances, such as the RF shielded room of an imaging system magnetic resonance imaging, in operating rooms near active AF surgical equipment, electrophysiology laboratories, armored rooms or areas where short wave therapy equipment is used.

- Do not use the system around strong electric field, electromagnetic field (e.g. MRI scan room) and mobile wireless communication devices. Using the device in an improper environment may cause malfunction or damage.
- The compliance with EMC and EMI regulation cannot be guaranteed by the use of modified cables or those which does not comply with the same standards under what the equipment was validated.
- The system must not be used adjacent or supported by other equipment. The recommendations of this manual must be followed.
- Do not use accessories, transducers, internal parts of components and other cables other than those previously specified by the manufacturer. This may result in increased emission or decreased electromagnetic immunity and result in improper operation.

- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30cm to any part of the ultrasound system, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- To maintain basic safety in relation to electromagnetic disturbances during the expected service life, always use the system in the specified electromagnetic environment and follow the maintenance recommendation described in this manual.

**The following tables provide information on compliance of the equipment according to the standard EN 60601-1- 2:2015.**

**Table 1 Compliance class**

Emissions Test	Compliance	Electromagnetic Environment and Guidance
RF emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	

Harmonic emissions IEC 61000-3-2	Class A	The equipment is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

**Table 2- Compliance standards**

Phenomenon	Basic Standard of	Immunity Test	Level of
Electrostatic discharge	IEC 61000-4-2	±8 KV contact ±2 KV, ±4 KV, ±8 KV,±15KV air	±8 KV contact ± 2 KV, ± 4 KV, ± 8 KV, ±15KV air
Radiated RF EM fields1	IEC 61000-4-3	3V/m 80 MHz-2.7 GHz 80% AM at 1 KHz	3V/m 80 MHz-2.7 GHz 80% AM at 1 KHz
Proximity fields from RF wireless communication equipment	IEC 61000-4-3	See tableS	ee table
Electrical Fast/Transients bursts	IEC 61000-4-4	±1 KV 100 KHz repetition frequency	±1 KV 100 KHz repetition frequency
Conducted disturbances induced by RF fields.	IEC 61000-4-6	3V 0.15 MHz-80 MHz 6 Vm in ISM bands between 0 .15 MHz and 80 MHz 80% AM at 1KHz	3V 0.15 MHz-80 MHz 6 Vm in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1KHz
Rated power frequency magnetic fields	IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz

**Table 3 - Test specifications for  
ENCLOSURE PORT IMMUNITY to RF  
wireless communications equipments**

Test Frequency (MHz)	Band (MHz)	Service	Modulation	Maximum Power (W)	Distance (m)	Immunity Test Level (V/m)
385	380-390	TETRA 400	Pulse modulation 18 Hz	1.80	.3	27
450	430-470	GMRS 460, FRS 460	FM ±5 KHz deviation 1KHz sine	2	0.3	28
710						
745	704-787	LTE 13, 17 Band	Pulse modulation 217 Hz	0.2	0.3	9
780						
810						
870	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE 5 Band	Pulse modulation 18 Hz	2	0.3	28
930						
1720	1700-1990	GSM 1800, CDMA 1900, GSM 1900, DECT, LTE 1, 3, 4 , 25 Band, UMTS	Pulse modulation 217 Hz	2	0.3	28
1845						
1970						
2450	2400-2570	Bluetooth, WLAN 802.11 b/g/n, RFID 2450, LTE 7 Band	Pulse modulation 217 Hz	20	.3	28
5240	5100-5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.29	0.3	
5500						
5785						

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Electronics Co.,Ltd**
**WellKang Ltd ([www.CE-marking.eu](http://www.CE-marking.eu))**