Terapio⁺

SMART CIRCULATION

Relieve Tired Legs, Feet & Body

Transcutaneous Electrical Nerve Stimulator

USER MANUAL

KTR-4015

Thanks for purchase of this product, and please
read this manual carefully before use of it!




ENGLISH

Read these instructions carefully and keep them for later use.



TABLE OF CONTENT

Content of packaging	2
Getting to know your device	4
Important notes	4
Step-by-step operating instructions	8
Overview of the placement of the electrodes/pads	9
Troubleshooting	11
Cleaning, storage and replacement parts	12
Warranty	13
Applied parts of the product	13
Technical Specifications	14
Safety Precautions	15
EMC statements	18
Symbols	18
Executive Standards	20
Appendix	20

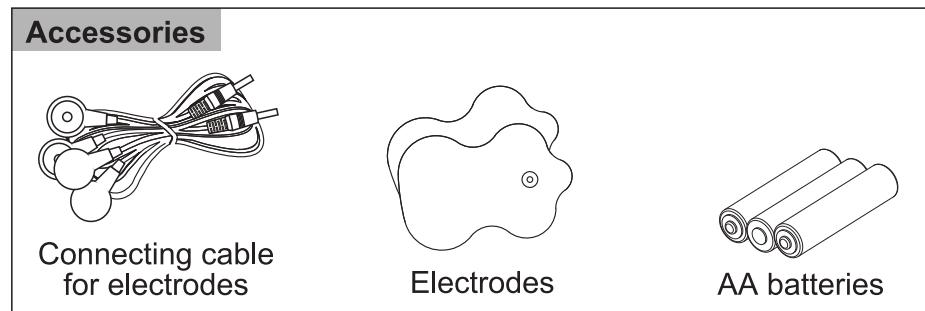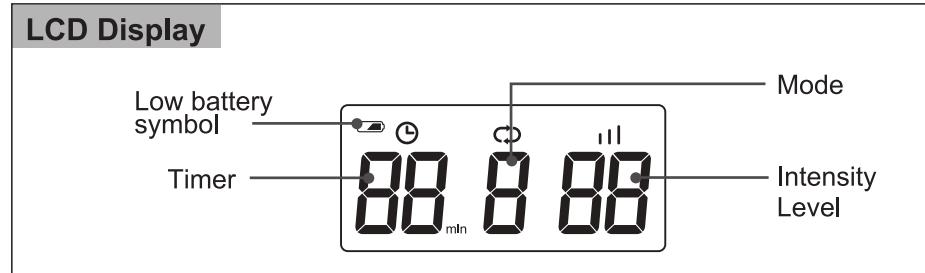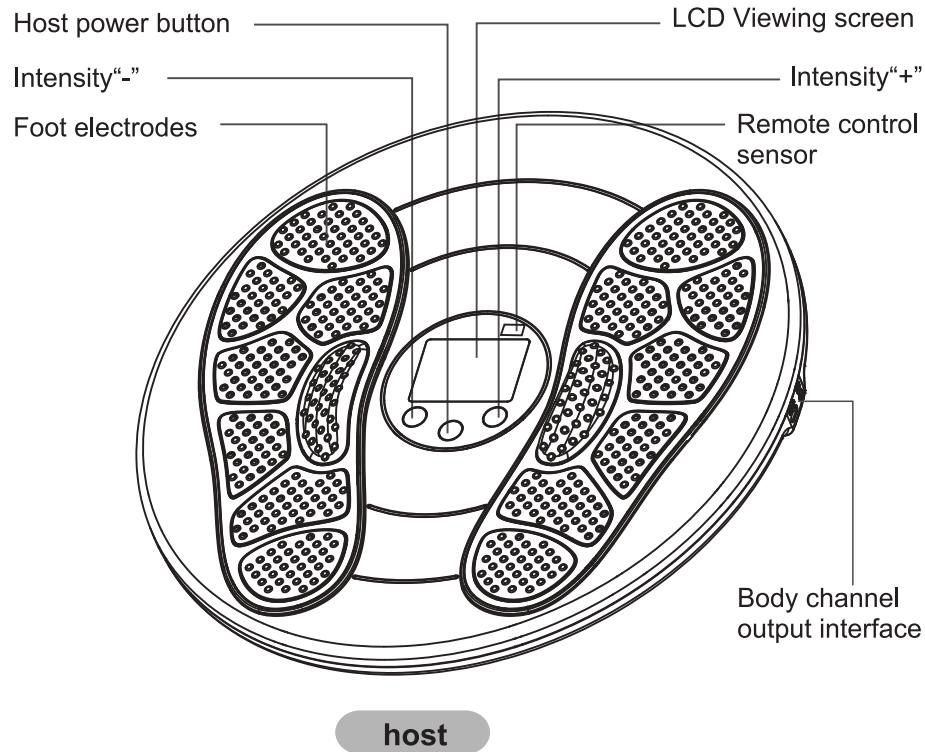Remote control (Choose accessories)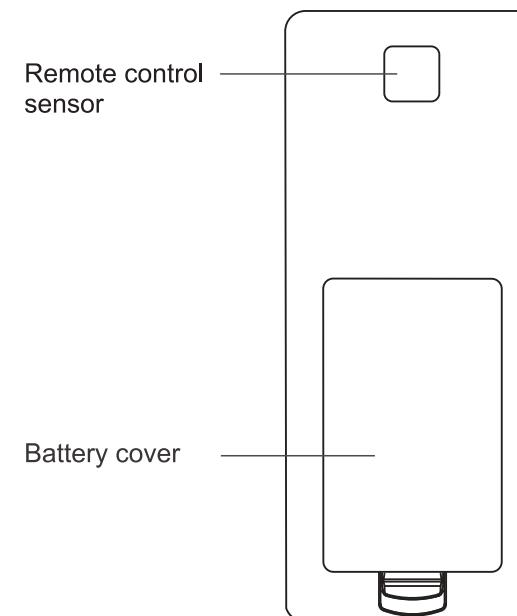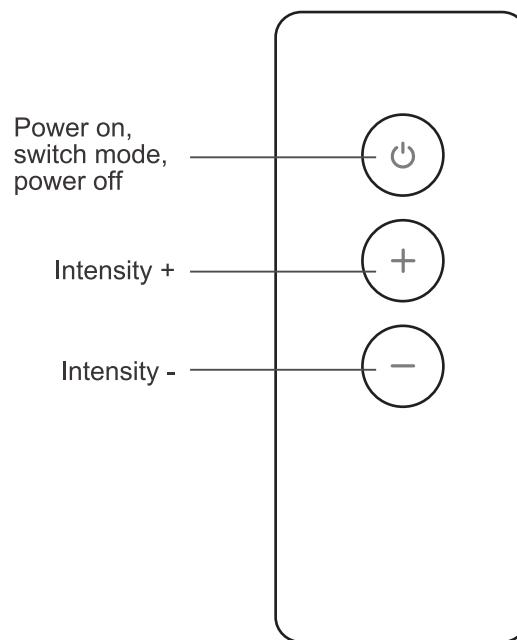

Getting to know your device

What is a Terapio Smart Circulation Pain Reliever and what can it do?

The Terapio Smart Circulation Pain Reliever unit is a medical device that falls into the electrostimulation device category.

It offers a wide range of functions for increasing general well-being, pain relief, maintaining physical fitness, relaxation, muscle revitalisation and combating tiredness.

TENS, or transcutaneous electrical nerve stimulation, relates to the electrical stimulation of the nerves via the skin. TENS is an effective non-pharmacological method of treating various types of pain that have a variety of causes. It has no side-effects if administered correctly. The method has been clinically tested and approved and can be used for simple self-treatment.

The pain-relieving or pain-suppressing effect is achieved by inhibiting the transference of pain to nerve fibres (caused mainly by high-frequency impulses) and by increasing the secretion of endorphins in the body. Their effect on the central nervous system reduces the sensation of pain.

The method is scientifically substantiated and approved as a form of medical treatment. Any symptoms that could be relieved using TENS must be checked by your GP. Your doctor will also give you instructions on how to carry out a TENS self-treatment regime.

Important notes

The device is not a substitute for medical consultation and treatment. Consult your doctor first if you are experiencing any pain or if you are suffering from an illness.

WARNING!

To avoid damage to health, we strongly advise against using this unit in the following situations:

- With implanted electrical devices (such as a pacemaker)
- In the case of metal implants

- If you use an insulin pump
- If you have a high temperature (e.g. > 39°C)
- If you have a known or acute cardiac arrhythmia, or disorders of the heart's impulse and conduction system
- If you suffer from a seizure disorder (e.g. epilepsy)
- If you are pregnant
- If you have cancer
- After an operation, if strong muscle contractions could affect the healing process
- The device must never be used near to the heart. The stimulation electrodes must not be placed on any part of the front ribcage (where the ribs and breastbone are located), especially not on the two large pectorals. This can increase the risk of ventricular fibrillation and induce cardiac arrest.
- On the skeletal skull structure, or around the mouth, throat or larynx
- In the neck/carotid artery area
- In the genital area
- On acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin—whether painful or not—reddened skin, rashes (e.g. allergies), burns, bruises, swellings, both open and healing wounds and post-operative scars where the healing process could be affected)
- In humid environments (e.g. in the bathroom)/when bathing or showering
- After consuming alcohol
- If connected to a high-frequency surgical device
- In the case of acute or chronic diseases of the gastrointestinal tract.

In addition, the stimulation should not be applied above or through the head, directly onto the eyes, covering the mouth, to the front of the neck (especially not to the carotid artery), or with the electrode surfaces placed on the chest and upper back or across the heart.

Before using the device, consult your doctor if any of the following applies to you:

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thromboembolic conditions or recurrent malignant growths
- Any skin conditions
- Unexplained chronic pain in any part of the body
- Diabetes
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders)
- If you are receiving medical treatment
- In the event of complaints linked to stimulation treatment

- If you suffer from persistently irritated skin due to long-term stimulation at the same electrode site

IMPORTANT

Only use this unit

- On adults
- For the intended purpose and as specified in these instructions for use. Improper use can be dangerous.
- For external use
- With the original accessories supplied, which can be re-ordered. Failure to do so invalidates the warranty.

PRECAUTIONS:

- Always pull firmly on electrodes to remove them from the skin to prevent injuries in the unusual case of highly sensitive skin.
- Keep the device away from sources of heat and do not use it in close proximity (approx. 1 m) to shortwave or microwave devices (e.g. mobile phones); doing so can result in unpleasant current peaks.
- Do not expose the device to direct sunlight or high temperatures.
- Protect the device from dust, dirt and humidity.
- Never immerse the device in water or other liquids.
- The device is suitable for self-treatment.
- For hygiene reasons, the electrodes may only be used on one person.
- If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.
- Switch off the unit or the respective channel first before removing or relocating electrodes to prevent unintentional stimulation.
- Do not modify electrodes (e.g. by cutting them).
- Do not use the device whilst asleep, driving a vehicle or operating machinery.
- Do not use whilst undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous.
- Ensure that no metallic objects (e.g. belt buckles or necklaces) come into contact with the electrodes during stimulation. If you are wearing jewellery or have piercings in the area to be treated (e.g. a navel piercing), these must be removed before using the device. Failure to do so could result in spot burns.

- Keep the device away from children.
- Do not entangle the electrode cables (including contacts) with headphones or other devices; do not connect the electrodes to other devices.
- Do not use the device whilst using other devices that transmit electrical impulses into your body.
- Do not use in the vicinity of highly flammable substances, gases or explosives.
- Do not use rechargeable batteries; always use the same battery types.
- During the initial few minutes, use the device while sitting or lying down in order to minimise the risk of injuries as a consequence of isolated cases of vagal responses (feeling of faintness). If you feel faint, immediately switch off the device, lie down and support the legs in an elevated position (approx. 5–10 min).
- Treatment of the skin with moisturising lotions or ointments beforehand is not recommended as this considerably increases the electrode wear and may cause unpleasant current peaks.
- This device is not intended for use by people with restricted physical, sensory (e.g. reduced sensitivity to pain) or mental skills or a lack of experience and/or lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device.
- If the adhesive capability of the adhesive electrodes decreases, replace them immediately. Only use the device again with new adhesive electrodes. Otherwise the unequal adhesion of the adhesive electrodes may lead to skin injuries. Replace the electrodes with new ones after having used them 20 times at most.

Damage

- If the device is damaged, do not use it; contact your retailer or the specified Customer Services address.
- In order to preserve the device's effectiveness, do not drop it or disassemble it.
- Check the device for signs of wear and tear or damage. If there are such signs, or if the device has been used improperly, it must be returned to the manufacturer or retailer before further use.
- Switch the device off immediately if it is faulty or not working properly.
- Do not attempt to open and/or repair the device yourself. Repairs may only be carried out by Customer Services or authorised retailers. Failure to comply will result in voiding of the warranty.
- The manufacturer is not liable for damage resulting from improper or careless use.

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Choking hazard! Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- Risk of explosion! Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.

Step-by-step operating instructions

Insertion/changing of the batteries

1. Open the battery compartment at the back of the unit.
2. Insert three AA batteries (1.5 V) into the battery compartment; make sure that the symbols (+/- match).
3. Close the battery compartment.
4. Do the same for the remote control, using the supplied two AAA batteries.

Switching the device on and off

Before using the unit, clean your feet with a damp towel. Make sure both feet are fully touching the electrodes, then completely relax and begin using the remote control.

1. Press and hold the power button to switch on the device.
2. The device turns off automatically after the treatment time has elapsed.
3. Press the power button on the unit or on the remote to switch off the device manually. The screen goes blank and the device turns off.

Selection of the function

The device offers 9 therapeutic modes.

1. Press the power button on the remote control or on the main unit to find a suitable program. Programs differ in terms of pulse width and frequency. Select a program that you prefer.
2. We recommend alternating between programs throughout your treatment.

Setting the treatment intensity levels


1. To adjust the intensity, press “+” to increase and “-” to decrease levels. There are 20 levels in total for adjustment. It.

Treatment duration

1. By default, a session lasts 30 minutes.
2. The unit turns off automatically after the treatment time has elapsed.
3. It is recommended to use the unit twice a day for 15-30 minutes to maximises benefits.

Overview of the placement of the electrodes/pads

The units comes with a pairs of reusable electrodes that can be used to deliver electrical stimulation to the knees, shoulders, back or arms.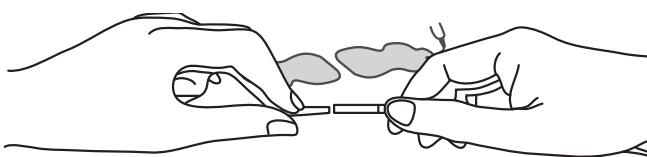

1. Connect the cable to the electrodes before placing them on the skin.
2. Plug the electrode cables into the unit.


Proper preparation of the skin that is covered by the electrodes allows a better stimulation of the target tissue, prolongs the life of the electrodes and reduces the risk of skin irritation.

After connecting the cable to the unit, prepare your skin at the spots where electrodes will be placed as follows:

1. Determine the spots where the electrodes are to be placed.
2. Wash this area with mild soap and water (do not use alcohol). Rinse and dry carefully.
3. Trim excess body hair in this area with a pair of scissors (do not shave).
4. Optionally, apply skin lotion to the area, to supply the skin with a protective


layer. Apply, let dry and attach the electrode as described in this manual. In this way you decrease the risk of skin irritation and prolong the life of your electrodes.

5. When you remove the electrodes, always pull them in the direction of the hair growth.
6. It may be helpful to apply skin lotion to the area of the electrode placement when you are not wearing the electrodes.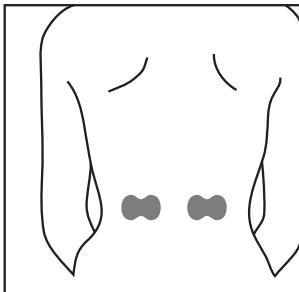


LOWER BACK

Place the unit vertically on each side of your back in the lower back area.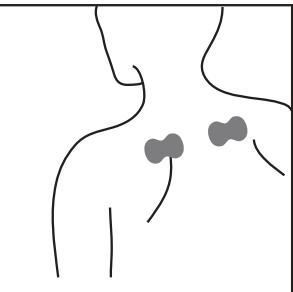


UPPER BACK

Place the unit vertically on each side of your back in the upper back area.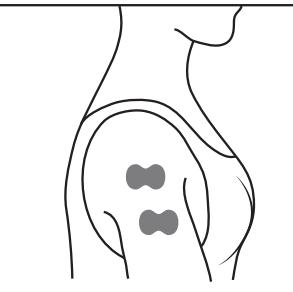


SHOULDER

Place the unit on the top part of your shoulder.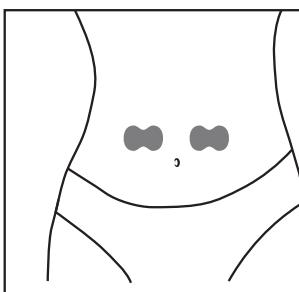


STOMACH MUSCLES

Place the unit vertically on your belly button.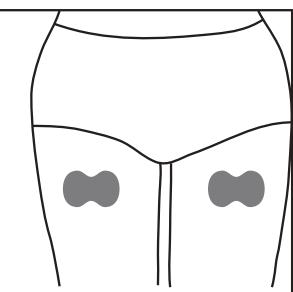



FRONT OF THE THIGHS

Place the unit vertically on the front of the thigh muscle.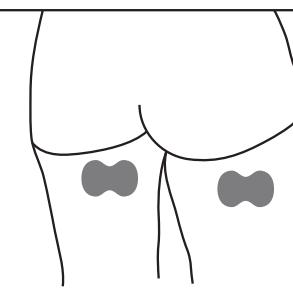

BACK OF THE THIGHS

Place the unit vertically on the back of the thigh muscle. Do not place pads too low on the leg, as this can lead to unpleasant contractions.

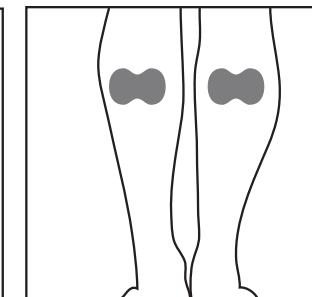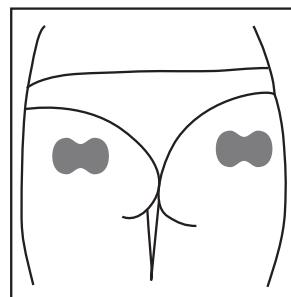CALF MUSCLES
Place the unit vertically on each calf muscle.
Do not place the pads too low on the leg, as this can lead to unpleasant contractions.

BUTTOCK

Place the unit vertically on the buttocks, halfway between the centre line and the side of your body.

Troubleshooting

Before use you should check the device and accessories to avoid damage or malfunction. Some simple checks are:

1. Make sure that the battery has sufficient power and that it is not corroded.
2. Ensure that the cables are firmly inserted into the socket device.

The table below shows some problems that may occur occasionally. If you are unable to solve the problem yourself, please contact your authorised dealer or customer service.

Problem	Cause	Solution
The device does not turn on	The batteries are not charged or running low	Insert a new battery
The device turns on but then turns off	The batteries are not inserted correctly	Reinsert the batteries or replace them
The device does not turn on, but generates electrical impulses	The batteries are flat; the cord is defective or not properly connected. The duration of the treatment has elapsed	Replace the batteries or cord; connect the cord properly. Switch off the device and then switch it back on again
The device does not turn on despite new batteries		Contact customer service
There is no or little tingling sensations	The electrodes may not be in contact with your feet or your body	Ensure electrodes are in good contact with your body. Consider wetting your skin with a damp towel
Red spots appear on the skin after use	In extremely rare cases, your skin may be allergic to the electrodes	Check your history of skin allergies and reduce the usage time or stop using the electrodes. Consider a desensitisation treatment

Cleaning, storage and replacement parts

Adhesive electrodes

- To ensure that the electrodes remain adhesive for as long as possible, clean them carefully with a damp, lint-free cloth or clean the underside of the electrodes under lukewarm, running water and pat dry with a lint-free cloth. Before cleaning with water, remove the connection cables from the electrodes.
- Reapply the electrodes to the carrier foil following treatment.

Cleaning the device

- Remove the batteries from the device before cleaning.
- Clean the device after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution.
- Do not use any chemical or abrasive cleaning agents. Ensure that no water enters the device.

Reuse of the device

Once it has been properly prepared, the device can be used again. Preparation includes replacement of the treatment electrodes as well as cleaning the surface of the device using a cloth moistened with a mild soapy solution.

Storage

- Remove the batteries from the device if you will not be using it for a prolonged period of time. Leaking batteries may damage the device.
- Do not make sharp bends in the connection cables and electrodes.
- Disconnect the connection cables from the electrodes.
- Reapply the electrodes to the carrier foil after use.
- Store the device and accessories in a cool, well-ventilated space.
- Never place any heavy objects on the device.

Replacement parts

Order original electrodes and replacement parts from terapio.com

Warranty

This product is guaranteed for a period of one year from the date of purchase against mechanical and electrical manufacturing defects. There are no serviceable parts inside this device. Any attempted repair by unauthorised persons invalidates the warranty. In the unlikely event that you experience a problem, please return the device to the retailer where you made the purchase, along with your receipt. This does not affect your statutory rights.

Contact customer support at terapio.com for any assistance.

Applied parts of the product

Applied parts of the product include foot and body electrodes.

Electrodes

The electrode is made of medical-grade hydrogel and carbon film, which is of biocompatibility.

Specification	Instructions
Dimensions	70x52 mm
Electrode impedance	$150\Omega \pm 10\%$
Service life	2 years
Service life Transport and storage environment	Temperature: 0~45°C Relative humidity: ≤80%RH Atmospheric pressure: 50~106kPa

Maintenance and cleaning

Electrodes should be always clean on their surfaces, without any dirt, oil, sticky substance, etc., otherwise, their viscosity will decrease. Dirty electrodes could be cleaned with clean water and then dried in air for a period of time of use.

Storage conditions

- Please keep the electrodes out of the reach of children.
- Please store them in a dry and well-ventilated place, and keep them away from direct sunlight, high temperature or humidity.
- Please never try to disassemble, repair or modify the electrodes, otherwise an accident or malfunction may occur.

Instructions on use of electrodes

- Please clean your skin thoroughly before each use of the electrodes, since the lotion, makeup or dirt left on the body skin will cause the poor application of electrodes.
- Please apply the electrodes to your body parts with a pain.
- Please shut down the electric pulse outlet before application of electrodes to your body skin.
- Please insert the circular plug of the electrode connecting cable into the electrode outlet on the product, and connect the other end of the cable to electrodes.
- Remove the protective film on electrodes and firmly apply them to your body parts with a pain. Keep the protective film removed into a bag for future storage of electrodes.

Accessories

Accessories include: one remote control, two electrodes, three AA batteries and two AAA batteries, and one electrode connecting cable.

All the accessories could be separated from the main unit. Please use electrodes attached with the product. Other types of electrodes are prohibited.

Technical Specifications

Product specifications

Product name	Transcutaneous Electrical Nerve Stimulator
Model	KTR-4015
Power supply	Main unit: 4.5V DC 100mA 3 AA batteries Remote control: 2 AAA batteries
Safety category	Type BF of class II
Dimensions of the main unit	350×350×68.5mm
Package Size	370×370×75mm
Net weight	Main unit: 1,180g / Remote control: 58g
Acceptable weight for foot electrode	≥50Kg
Software version No.	A/0
Product service life	3 years

Ambient conditions

	Temperature	Humidity	Barometric pressure
Normal operation	+5°C~+40°C	15%~93%RH	700hPa~1060hPa
Storage	-20°C~+70°C	0~93%RH	700hPa~1060hPa
Transport	-20°C~+55°C	15%~93%RH	700hPa~1060hPa

Electronic performance

Pulse frequency	1Hz~100Hz
Pulse width	100μs~400μs
DC component	0V
Peak output voltage	5~80V
Power of a single pulse at the maximum output amplitude	>7μC
Maximum output energy of a single pulse	≤300mJ
Maximum output amplitude RMS	<10V
Peak output voltage during open circuit measurement	≤500V
Output amplitude adjustment	Be able to withstand the effects of open and short circuits at output terminal, with no degrading in performance.
Adjustment of output amplitude	Continuous and uniform, keeping the minimum output not greater than 2% of the maximum output
Rated load impedance	500Ω, with ±10% error
Set treatment time	15/30/45/60minutes, with an acceptable error of ±10%

Describe for software

Software name: KTR-4015-2081A V1.0
Version: V1.0

Safety Precautions

- Warning signs and symbols referred in this manual are for safe and proper use of the product by users, thus preventing damages to users or others.

Warning signs, symbols and their meanings are as follows:

Caution signs	Meaning
	Indicates that when used incorrectly, it may result in death or serious injury to a person.
	Indicates that when used incorrectly, it may result in death or serious injury to a person.
	Indicates that when used incorrectly, it may result in death or serious injury to a person.
Symbols	
	Means prohibition (something that must be prohibited). Specific items prohibited are provided within or near this symbol in text or graphics. The warning sign on the left represents "removal prohibited"
	• means constraint (something that must be followed). Specific items constrained are provided within or near this symbol "●" in text or graphics. The warning sign on the left represents "General Constraints".

Contraindications	
<p>1. The product is prohibited from being used to patients with bleeding tendency, local acute suppurative inflammation, local malignant tumor, local pacemaker, local metal implants or skin sensory disorders, Patients with deep venous thrombosis, and to human hearts and the lower abdomen of pregnant women;</p> <p>2. Please never use the product with any other electronic medical device, such as the life-sustaining medical electronic devices of pacemaker, artificial heart-lung machine, and other medical electronics such as the electrocardiograph;</p> <p>3. A burn may occur to the patient's body part applied with the electrode when the product, together with a high-frequency surgical equipment, is used to the patient, which also may damage the product; the product may be instable in output when used near a short-wave or microwave therapy equipment (1 meter);</p> <p>4. Patients with heart disease;</p> <p>5. Poisoning patients or others with high fever;</p> <p>6. Infants and children, and those without the ability or consciousness to express;</p> <p>7. Use of electrodes in close to your chest will increase the risk of fibrillation.</p>	

Warning	
<p>1. Please consult the doctor for pregnant women and those in Menstrual period, and patients with sensitive skin, heart disease, abnormal blood pressure, cancer, cerebrovascular disease, acute disease and those undergoing treatment, Patients with deep venous thrombosis, before use of the product.</p> <p>2. Unless medically suggested by an expert, patients with an electronic device implanted (such as a pacemaker) are not recommended for stimulation with the product.</p> <p>3. The product isn't allowed for those with skin perception disorders.</p> <p>4. It is prohibited from using the product in bathing, sweating, or sleeping.</p> <p>5. Patients with cerebral hemorrhage: prohibited in their unstable conditions; those with sequelae must be supervised by a doctor in use of the product.</p> <p>6. The product is prohibited for those with purulent inflammation, acute blood poisoning or persistent high fever.</p> <p>7. The product is also prohibited for those with an acute cardiovascular and cerebrovascular disease.</p> <p>8. It is prohibited for those with bleeding tendency, acute purulent inflammation or metal implants to use the product.</p> <p>9. Instruments and accessories used by more than one patient need to be cleaned and disinfected by rinsing with a wet cloth.</p>	
Warning	
<p>1. Please don't use the product near your heart and to your head, eyes, front waist (especially carotid artery), lower back, mouth, genitals and body parts with skin diseases.</p> <p>2. Please don't move the product during use. When changing the body part of treatment, please power off the product first before restarting it, otherwise the stimulation may be strong.</p> <p>3. Please don't use the product to children or those without the ability or consciousness to express.</p> <p>4. Please stop using the product immediately and then seek medical advice in case of any discomfort during use</p> <p>5. Please unplug the product after use or when not in use.</p> <p>6. Please never use the product with any other electronic medical device, such as the life-sustaining medical electronic devices of pacemaker, artificial heart-lung machine, and other medical electronics such as the electrocardiograph, otherwise a danger may occur.</p> <p>7. Please don't use the product in places with high temperature, combustibles, electromagnetic radiation, or humidity.</p> <p>8. Please don't disassemble, repair or modify the product, otherwise a malfunction or electric shock may occur.</p> <p>9. Please stop using the product immediately and then seek medical advice in case of any skin discomfort during use of the product.</p> <p>10. Please don't discard the product at will at the end of its service life. Instead, please follow the local laws and regulations to dispose it. You have to contact the local authority for proper ways to dispose the potentially bio-hazardous components and accessories.</p>	

EMC Statements

1. The product requires special attention to EMC and requires to be installed. Please use it based on the EMC information provided, since it may be affected by portable and mobile RF communication devices.
2. The product has been thoroughly tested and inspected to ensure its excellent performance and proper operation.

Note: the product should not be used near or on any other device; if it is necessary for this, please observe if it can work normally.

⚠ Warning: using accessories other than those specified may result in increased emissions or decreased immunity of the product. Guidance and Manufacturer's declaration can be seen in the attachment.

Symbols

NOTATION	MEANING
LOT	Batch/lot number
SN	Serial No.
	Manufacturer
	European authorised representative
	Date of manufacture
	Warning
	Warning on danger
MD	Medical device

	Type BF applied part
	It means prohibition (something that must be prohibited)
	It means compulsory execution (something that must be followed)
	Indoor use only
	No sun exposure
	No rainwater
	"WEEE (Waste Electrical and Electronic Equipment)". Wastes shall be disposed in accordance with laws.
	CE Mark and Agency Code
	Standby
	Please refer to the user manual
	Temperature limitations. It indicates the range of temperature to which medical devices can be safely exposed. The upper and lower temperature limits shall be indicated near the upper and lower horizontal lines.
	Humidity limits. It indicates the range of humidity to which medical devices can be safely exposed. The humidity limits shall be indicated near the upper and lower horizontal lines.
	Atmospheric pressure limits. It indicates the range of atmospheric pressure to which medical devices can be safely exposed. The atmospheric pressure limits shall be indicated near the upper and lower horizontal lines.
	Direct Current
	Electrodes
IP21	Dustproof and waterproof level, to avoid the falling of solid objects larger than 12.5mm and the vertical penetration of water to cause adverse effects.

Executive Standards

The product should be in accordance with the following standards and regulations:

1. Medical electrical equipment - Part 1: General requirements for basic safety and essential performance (EN 60601-1:2006 + A1:2013)
2. Medical electrical equipment - General requirements for basic safety and essential performance - Collateral standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment (EN60601-1-11:2015)
3. Medical electrical equipment - Part 2-10: Particular requirements for the safety of nerve and muscle stimulators (EN60601-2-10:2015)
4. Medical electrical equipment - Part 1-2: General requirements for safety - Collateral standard: Electromagnetic compatibility - Requirements and tests (EN60601-1-2:2015)

Appendix

Guidance and manufacturer's declaration – electromagnetic emission		
The Transcutaneous Electrical Nerve Stimulator is intended for use in the electromagnetic environment specified below. The buyer or user of the Transcutaneous Electrical Nerve Stimulator should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Transcutaneous Electrical Nerve Stimulator use RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference to nearby electronic equipment.
RF emission CISPR 11	Class B	The Transcutaneous Electrical Nerve Stimulator is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	N/A	

Guidance and manufacturer's declaration – electromagnetic immunity			
The Transcutaneous Electrical Nerve Stimulator is intended for use in the electromagnetic environment specified below. The buyer or user of Transcutaneous Electrical Nerve Stimulator should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+8kV contact +2kV, + 4kV. +8kV, + 15kV air	+8kV contact +2kV, + 4kV. +8kV, + 15kV air	Floors should be wood, concrete or ceramic tile. If floor is covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1kV line(s) and neutral	± 1kV line(s) and neutral	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 s	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Transcutaneous Electrical Nerve Stimulator requires continued operation during power mains interruptions, it is recommended that the Transcutaneous Electrical Nerve Stimulator be powered from an uninterruptible power supply.
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Recommended separation distances between portable and mobile RF communications equipment and the Transcutaneous Electrical Nerve Stimulator.

The Transcutaneous Electrical Nerve Stimulator is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The buyer or user of the Transcutaneous Electrical Nerve Stimulator can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Transcutaneous Electrical Nerve Stimulator as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 KHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For rated maximum output power of transmitter that's not listed in the above chart, the recommended isolation distance d (unit is meter) can be determined by equation in the corresponding transmitter frequency column. P is the maximum output rated power of transmitter provided by its manufacturer, the unit is watt (W).

NOTE 1 When under 80MHz and 800MHz, use equation of higher frequency.

NOTE 2 These guidelines couldn't possibly fit all situations. Electromagnetic transmission is affected by buildings, objects, the absorption and reflection of human body.

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Shenzhen Kentro Medical Electronics Co.,Ltd
WellKang Ltd (www.CE-marking.eu)